MAJ le 18/09/2014

ALOVA

®

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User manual

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WINCARE
g r o u p

ASKLÉSANTÉ

FR

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GB

You have just bought a product from the ALOVA range. Thank you for making this purchase. Please read these instructions for use carefully to keep it in optimum condition.

Pillows or Accessories - Setting up and Use:

Remove their packaging

Follow the pictures below for proper use of your pillow or accessory..

Alova Cervical
Contour Pillow

Alova Classic
Pillow

Alova Lumbar
Support

Alova Half-moon
Cervical Pillow

Alova Back
Support

T1, T2 & T3
Alova Cervical Pillow
+
Alova Travel
Cervical Pillow

Cleaning – maintenance :

Removable covers (100% polyester).

	Machine wash at 40°C. Normal mechanical action. Normal rinse. Normal spin-dry.
	Do not bleach
	Iron at a maximum temperature of 110°C. (Do not iron with steam)
	All solvents, except trichloroethylene, may be used, as per a normal procedure. Dry-cleaning possible
	Tumble-drying possible at medium temperature (60°C maximum)

Item treated with a biocidal substance that is safe for the user. www.winncare.fr

Fire :

ALOVA products comply with the requirements of decree No 2000-164 of 23 February 2000, guaranteeing the safety of persons against the risks of fire on any bed accessory..

Storage :

Store the products flat, away from direct sunlight and in a dry place. The Alova Travel Cervical pillow may be carried rolled up in the cover provided..

Disposal :

Dispose of your mattress and pillows/accessories in accordance with local regulations.

Warranty :

The pillows and accessories have a 3-year warranty ,excluding the cover (1 year)

This warranty begins from the date of purchase of the product at your wholesaler.

The warranty does not cover normal changes in hardness or properties that do not affect the pressure-relieving properties.

The warranty is valid only if the Alova product is used and handled as described in these instructions for use

If a claim is made under the warranty, you must present an invoice or sales receipt as documentation of your purchase along with the returned warranty slip.

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