Full slings

Universal

Universal Head Support

Universal Mesh

Universal Mesh Head Support

Universal Deluxe

All day

All day Head Support

Comfort

Comfort Mesh

EN: Technical user guide

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SE: Teknisk bruksanvisning

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Important

This manual must be read before the sling is used. This manual is intended for therapists and other personnel, in connection with the use and maintenance of *Ergo* slings.

Purpose

Ergolet's *Ergo* slings can be used for transfers in home care, nursing homes, institutions and private homes. *Ergo* slings are used to transfer people with physical or reduced mobility issues and for transfers from bed to chair, from chair to bathing and toilet chair etc.

Check list before use

It is the carer's responsibility to check the sling before use.

- It must be ensured that the sling does not have any visible damages or wear and tear and that the sling is clean.
- When the sling is used, qualified personnel must be present.
- The capacity written on the sling must never be exceeded!

Lifetime

The sling can be washed 80-100 times, which corresponds to once a month in 6-8 years. The product's lifetime is expected to be 5-10 years, if service checks and the stated maintenance instructions have been followed. It is the owner's responsibility that the annual inspection is performed.

Materials

The slings are produced in durable materials and tolerate the necessary washing. All slings are produced in polyester.

Label symbols

Declaration of Conformity

Recommended 92°C for washing

Do not bleach

Do not iron

Do not dry clean

Tumble dry at low temperature

Max 275kg


Maximum lifting capacity is 275kg

Placement of sling straps

Colour coded straps

The straps on most of the slings are marked with colours (red, yellow, green, blue), which makes adjustment of position and placement of the sling easier.

Corresponding sling straps must be attached to the spreader bar with the same colour. The sitting position can be varied by choosing different colours for the leg and shoulder straps.

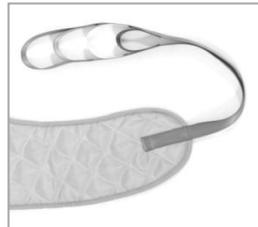Patented Ergolet sling hook system

Ergolet does not take any responsibility if the sling is placed incorrectly. It is always the carer's responsibility that the correct sling has been chosen and the sling is fitted correctly.

1. The sling strap is placed on the open hook (1).
2. The strap is pulled downwards and the hook will close (2).

The design of the lifting hook is such that the weight of the user will help close the hook.



To release the strap after lifting simply pull upwards to open the hook.

Note: Ensure that client's fingers cannot get trapped.

Other hooks

The fig A below shows placement of the strap on hooks of Pallas/150 stand aid.


The fig B illustrates placement of the straps on the hooks of Melas/150 mobile lift. The picture shows one leg strap and one shoulder strap. The other leg strap and shoulder strap should be placed on the other side of the spreader bar on the corresponding hook.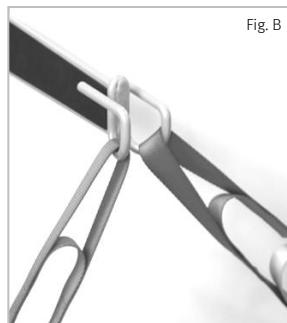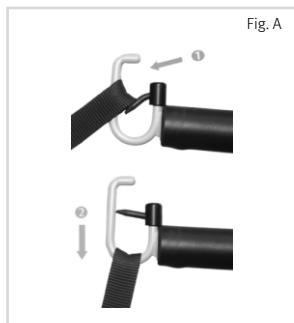

Strap color code

RED	SMALL
YELLOW	MEDIUM
GREEN	LARGE
BLUE	X-LARGE

Size marker

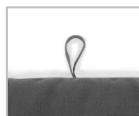Lifting

Lift from sitting position

The sling is placed under the user as shown on fig. 1. Lift the leg up and pull the leg straps under it. See fig. 2. The straps on the leg straps are crossed (fig 3) and placed on the hooks on the spreader bar.

Note: Only for Comfort sling:

The straps on the comfort sling are not supposed to be crossed. The short strap (opposite the yellow strap) is put through the yellow strap and both of the long straps are put through the short blue straps.

- Ensure that sling loops are placed correctly on the spreader bar hooks.
- Ensure the patient sits/lies safely before taking off the sling.

Above: Lift in the Ergolet Universal sling

Fig. 1

Fig. 2

Fig. 3

Lift from bed or floor

The user is rolled on to the side and the sling folded on the middle. The folded edge is placed under the user. See fig. 4. The user is rolled on to the other side and the sling is folded out again. The leg straps are put under the thighs of the user. Sling straps are placed on the hooks on the spreader bar. The user is being lifted (fig 5).

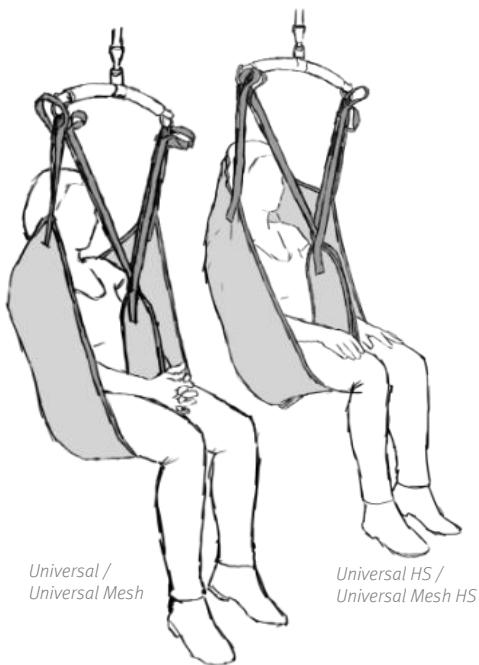
Fig. 4

Fig. 5

Universal sling

USABILITY

Universal sling for most transfer situations such as from bed to chair, from chair to wheel chair etc., when using either a mobile lifter or a ceiling hoist.



The model with head support can be used when the user has little or no head control. The mesh version dries quickly and can be used for shower situations.

Models


Universal Sling
Universal Sling Head Support
Universal Sling Mesh
Universal Sling Mesh Head Support

NB: The leg straps must be crossed – put one strap through the other.

TECHNICAL SPECIFICATION	
Max capacity	275 kg
Max washing temperature	92° C
Max drying temperature	55° C
Spreader bar compatibility	2-, 3- and 4-point
Sling is available in sizes	XS - XXL

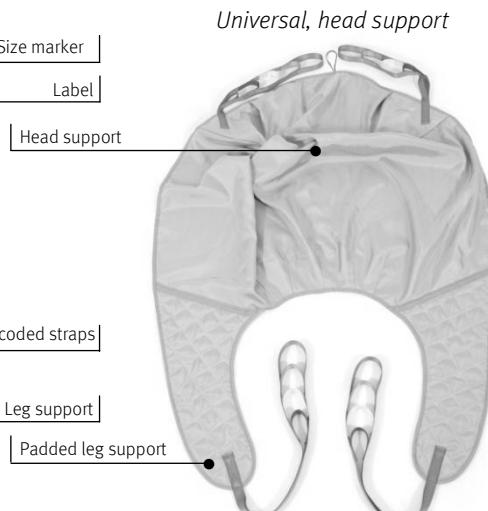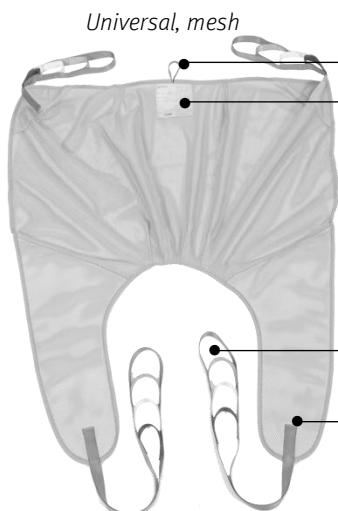Universal Deluxe sling

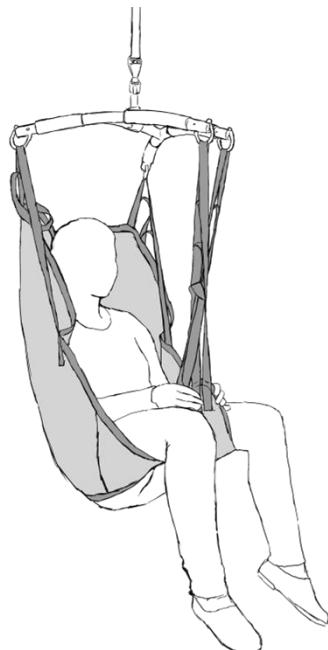
USABILITY

Universal Deluxe sling for most transfer situations such as from bed to chair, from chair to wheel chair etc., when using either a mobile lifter or a ceiling hoist.

NB: The leg straps must be crossed – put one strap through the other.

TECHNICAL SPECIFICATION	
Max capacity	275 kg
Max washing temperature	92° C
Max drying temperature	55° C
Spreader bar compatibility	3- and 4-point
Sling is available in sizes	XXS - XXL

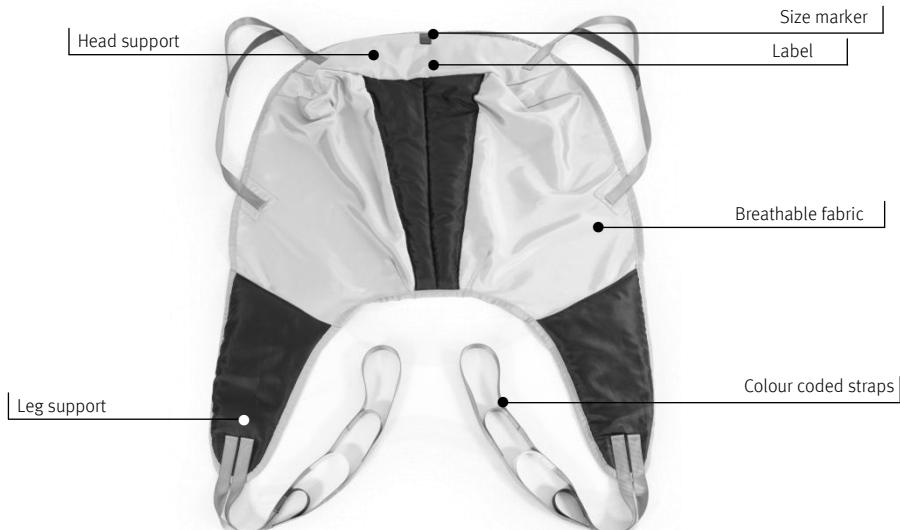All Day sling

USABILITY

For most transfer situations such as from bed to chair, from chair to wheel chair etc., when using either a mobile lifter or a ceiling hoist.


The model with head support can be used when the user cannot support the head.

The sling is made from breathable fabric, allowing the skin to breathe easily when sitting on the sling for longer periods.


NB: The leg straps must be crossed – put one strap through the other.

TECHNICAL SPECIFICATION	
Max capacity	275 kg
Max washing temperature	92° C
Max drying temperature	55° C
Spreader bar compatibility	2-, 3- and 4-point
Sling is available in sizes	XS - XL

*All Day
(w/o head support)*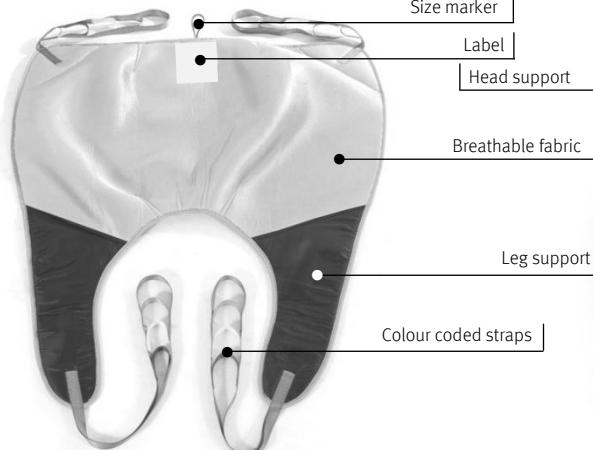

*All Day
(head support)*

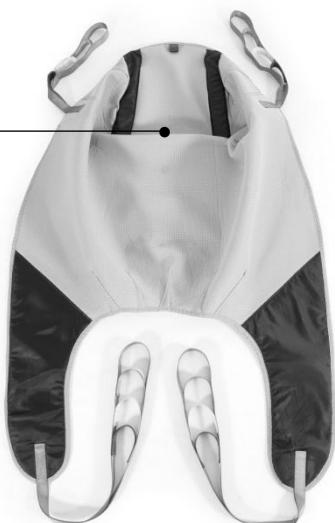Comfort sling

USABILITY

For transfers of users with reduced physical functions or amputation. Leg support with large surface distributes pressure on the upper leg.

For transfers from chair, shower chair, bed or floor.

The mesh version can be used in a bathing situation.

TECHNICAL SPECIFICATION	
Max capacity	275 kg
Max washing temperature	92° C
Max drying temperature	55° C
Spreader bar compatibility	2-point
Sling is available in sizes	XS - XXL

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